ClinCal® - Calibrator

Plasma Calibrator lyophilised / Plasma-Kalibrator lyophilisiert

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Intended use:

ClinCal® Plasma Calibrators are used for calibration of your analysis-system. This lyophilised calibrator is based on human plasma and is intended for a single-point calibration. After reconstitution the calibrator has to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 3.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

This product will be stable until the expiration date when stored unopened at 2 - 8 °C. After reconstitution the stability of the analytes is:

- 5 hours at 15 - 30 °C
- 1 day at 2 - 8 °C
- 10 days at below -18 °C
(avoid repeated freezing and thawing)

Notes:

The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinCal® Calibrators have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 0.94 %.

Mean values:

The mean values have been established in independent reference laboratories with supervision of RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK), with statistical methods.

Pack size:

ClinCal® Plasma Calibrator
5 x 3 ml, **order no.: 10213**

Precautions:

The human plasma which was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV1/2-, HBV- and HCV-antibodies, Hepatitis B-surface antigen, HIV- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the plasma calibrator should be considered as potentially infectious and treated with appropriate care.

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Technique of Analysis / Analysentechnik:

HPLC: Amino Acid Analyser (AAA) / Aminosäureanalysator (ASA)

RECIPE CHEMICALS + INSTRUMENTS GmbH
Dessauerstr. 3 D-80992 Munich / Germany
Tel.: +49 / 89 / 54 70 81 - 0 Fax: +49 / 89 / 54 70 81 - 11
Internet: www.recipe.de e-mail: info@recipe.de