ClinCal® - Calibrator

Plasma Calibrator lyophilised, for Lamotrigine, Sulthiame /

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The mean values have been assigned with the ClinRep® Complete Kit for Lamotrigine and Sulthiame. The uncertainty of the mean values is ± 1.0 % for both analytes at a confidence level of 95 %. The mean value of Lamotrigine is traceable to NIST-SRM900a.

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Intended use:

ClinCal® Plasma Calibrators are used for calibration of the HPLC-system. After reconstitution this lyophilised calibrator has to be prepared like a patient sample.

Reconstitution:

Add exactly 3.0 ml HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised plasma calibrator is stable for 48 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is:

- 12 hours at 15 - 30 °C
- 7 days at 2 - 8 °C
- 3 months below -18 °C (avoid repeated freezing and thawing)

The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 1.30 %.

Pack size:

ClinCal® Plasma Calibrator for Lamotrigine and Sulthiame, lyophilised: 1 x 3 ml, **order no.: 15113**

Caution:

The human plasma which was used for manufacturing the calibrator was tested for the following infectious markers and found negative: HIV1/2-, HBV- and HCV-antibodies, Hepatitis B-surface antigen, HIV1- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the plasma calibrator should be considered as potentially infectious and treated with appropriate care.

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