ClinCal® - Calibrator

Urine Calibrator lyophilised, for 5-ALA / PBG

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Intended use:

ClinCal® Urine Calibrators are used for calibration of the analytic system. After reconstitution this lyophilised calibrator has to be prepared like a patient sample.

Reconstitution:

Add exactly 5.0 ml HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored below -18 °C in the dark, the lyophilised urine calibrator is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is when stored in the dark:

- at least 24 hours when stored at 15 - 30 °C
 - at least 3 days when stored at 2 - 8 °C
 - at least 3 months when stored below -18 °C
- (avoid repeated freezing and thawing)

The variation of the filling volume (CV) is < 1 %. The average residual moisture of this lot is 0.39 %.
The creatinine concentration of this calibrator is 0.24 g/l.

Pack size:

ClinCal® Urine Calibrator for 5-ALA / PBG: 5 x 5 ml, **order no.: 17213**

Caution:

This ClinCal® Calibrator was manufactured from human urine. Therefore the material has to be considered as potential infectious and should be treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH