ClinTest® - Standard

Test Solution for β-Carotene, lyophilised /

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Intended use:

ClinTest® Test Solutions are intended to check the daily performance of your HPLC - system. This Test Solution is lyophilised and needs to be reconstituted before usage. Afterwards inject 100 µl directly into your analysing system. After separation, please compare the retention times of the peaks with those of earlier analyses. If you run the Test Solution for the first time, you can compare your results with the chromatogram on the back of this data sheet. Besides retention times you are also able to check peak shape, integration and baseline with your test chromatogram.

Reconstitution:

Add exactly 3.0 ml of methanol to the vial and mix well.

Storage and stability:

Originally closed and stored below -18 °C in the dark, the lyophilised Test Solution is stable for 36 months, but not longer than the expiration date printed on the label. After reconstitution the Test Solution is stable for at least 6 weeks, when stored below -18 °C and in the dark.

Pack size:

ClinTest® Test Solution for β-Carotene, lyophilised: 3 ml, **order no.: 30014**

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RECIPE CHEMICALS + INSTRUMENTS GmbH