ClinChek® - Control
CSF/Serum Control lyophilised /
Liquor-/Serumkontrolle lyophilisiert

FOR CSF/SERUM QUOTIENTS /
FÜR LIQUOR/SERUM-QUOTIENTEN

Intended use:

ClinChek® CSF/serum controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on native human cerebrospinal fluid (CSF) and serum and are available with mean values and CSF/serum quotients in the normal as well as in the pathological range. After reconstitution the controls have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 1.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the lyophilised CSF/serum control is stable for 24 months, but not longer than the expiration date printed on the label. After reconstitution the stability of the analytes is:

- at least 24 hours when stored at 15 - 30 °C
- at least 14 days when stored at 2 - 8 °C
- at least 3 months when stored below -18 °C (avoid repeated freezing and thawing)

Notes:

The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume is max. +/- 1 %. The average residual moisture of the serum controls of this lot is 0.98 %.

Mean values:

The mean values and confidence intervals were established in certified educational laboratories of the German Association for Liquordiagnosics and Clinical Neurochemistry e.V. with supervision of RECIPE, according to the guideline of the German Medical Council for quality assurance (RiliBäk), with statistical methods. For value assignment nephelometric methods were used.

Pack size:

ClinChek® CSF/Serum Control
 CSF Level I, II and Serum Level I, II
 4 x 2 x 1 ml, **order no.: 8605**

Precautions:

The human CSF which was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV1- and HCV-RNA, HBV-DNA (NAT). The human serum which was used for manufacturing the controls was tested and found negative additionally for: HIV1/2-, HBV- and HCV-antibodies, hepatitis B-surface antigen. Nevertheless, the CSF/serum controls should be considered as potentially infectious and treated with appropriate care.

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RECIPE CHEMICALS + INSTRUMENTS GmbH