ClinChek® - Control

Serum Control liquid in ampules / Kontrollserum flüssig in Ampullen

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Application:

ClinChek® Controls are used for internal quality control in clinical chemical laboratories. These ready-to-use controls are based on human serum and are available in five different ranges of concentration. The controls have to be prepared like patient samples in one series of analyses.

Storage and stability:

Originally closed and stored at 2 - 8 °C, the material is stable for 36 months, but not longer than the expiration date printed on the label. After opening the ampules, the serum has to be analyzed immediately. The ampules are intended for single use only.

Notes:

Routine analyses of the controls are performed according to the test procedure for patient samples. The concentrations of the analyte are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek® controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each ampule of one lot. The variation of the filling volume is max. +/- 0.5 %.

Mean values:

The mean values and confidence intervals have been established in independent reference laboratories with supervision of an institution that is recommended by the guideline of the German Medical Council for quality assurance (RiliBäk).

Pack size:

ClinChek® Serum Control
Level I
10 x 2 ml, **order no.: 8890**

Level II
10 x 2 ml, **order no.: 8891**

Level III
10 x 2 ml, **order no.: 8892**

Level IV
10 x 2 ml, **order no.: 8893**

Level V
10 x 2 ml, **order no.: 8894**

Precautions:

The human serum which was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, hepatitis B-surface antigen, HIV1- and HCV-genome (PCR). Nevertheless, the controls should be considered as potentially infectious and treated with appropriate care. In addition, the serum contains 0.01% potassium cyanide as preservative. Do not pipet by mouth.

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In Germany Ethanol is subject to the guideline of the German Medical Council for quality assurance.

For assessment of the internal quality control, please also refer to table B 1a of the guideline. /

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RECIPE CHEMICALS + INSTRUMENTS GmbH