## ClinCal® - Calibrator

**Serum Calibrator lyophilised, for 25-OH-Vitamin D2/D3**

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The mean values have been assigned with the ClinRep® Complete Kit for 25-OH-Vitamin D2/D3. The uncertainty of the mean values is  $\pm 1.7\%$  for 25-OH-Vitamin D3 and  $\pm 2.1\%$  for 25-OH-Vitamin D2 at a confidence level of 95 %. The mean values are traceable to NIST-SRM972a.

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### Intended use:

ClinCal® Serum Calibrators are used for calibration of the HPLC-system. After reconstitution this lyophilised calibrator has to be prepared like a patient sample.

### Reconstitution:

Add exactly 2.0 ml HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

### Storage and stability:

This product will be stable until the expiration date when stored unopened below  $-18\text{ }^{\circ}\text{C}$ . After reconstitution the stability of the analytes is:

- 1 day at  $15 - 30\text{ }^{\circ}\text{C}$
- 7 days at  $2 - 8\text{ }^{\circ}\text{C}$
- 90 days below  $-18\text{ }^{\circ}\text{C}$  (avoid repeated freezing and thawing)

The variation of the filling volume (CV) is  $< 1\%$ . The average residual moisture of this lot is  $1.11\%$ .

### Pack size:

ClinCal® Serum Calibrator for 25-OH-Vitamin D2/D3: 5 x 2 ml, **order no.: 35013**

### Caution:

This ClinCal® Calibrator was manufactured from pooled calfserum. It should be considered as potential infectious and treated with appropriate care.

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