ClinTest® - Standard

Standard Solution for Hippuric Acids /

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Analyte / Analyt	Concentration* / Konzentration*
Hippuric Acid / Hippursäure	25.0 mg/l (140 µmol/l)
o-Methyl-Hippuric Acid / o-Methyl-Hippursäure	10.0 mg/l (51.8 µmol/l)
Internal Standard / Interner Standard	10.0 mg/l (72.4 µmol/l)
m-Methyl-Hippuric Acid / m-Methyl-Hippursäure	10.0 mg/l (51.8 µmol/l)

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Intended use:

ClinTest® Standard Solutions are intended to check the daily performance of your HPLC - system. This Standard Solution needs no preparation and can be injected directly into your analysing system (injection volume: 20 µl). After separation, please compare the retention times of the peaks with those of earlier analyses. If you run the Standard Solution for the first time, you can compare your results with the chromatogram on the back of this data sheet. Besides retention times you are also able to check peak shape, integration and baseline with your standard chromatogram. For calibration the ClinCal® Urine Calibrator, order no.: 9922 has to be used.

Storage and stability:

Stored at 2 - 8 °C, the unopened Standard Solution can be used until the expiry date indicated on the product label. After use of the Standard Solution, the bottle must be closed tightly and stored at 2 - 8 °C, to ensure that it can be used until the expiry date indicated on the product label.

Pack size:

ClinTest® Standard Solution for Hippuric Acids in Urine: 1 x 3 ml, **order no.: 52011**

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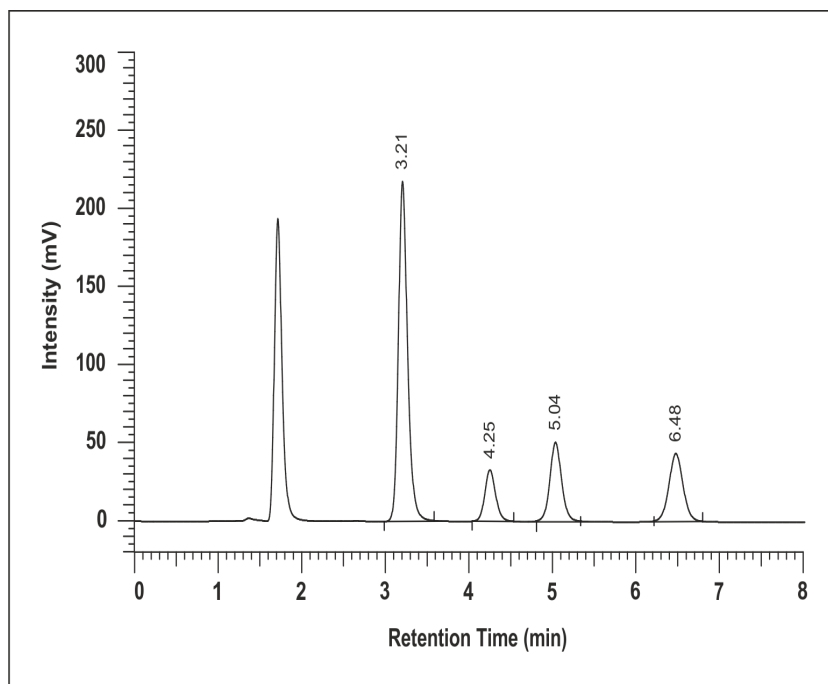
Hippuric Acid / Hippursäure	3.21 min
o-Methyl-Hippuric Acid / o-Methyl-Hippursäure	4.25 min
Internal Standard / Interner Standard	5.04 min
m-Methyl-Hippuric Acid / m-Methyl-Hippursäure	6.48 min

CE